spophy

MLI: 0307_2024

COOLANT SPRAY

ENG – Coolant Spray APLICATION AND INSTRUCTION

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www.spophy.com

ENG

General Description: Spophy Coolant Spray is a cooling spray for quick pain relief associated with muscle bruises, joint sprains, muscle cramps, and other injuries resulting from impacts, falls, or sprains, and for reducing pain during minor procedures such as venipuncture before inserting a cannula or catheter into a vein. It contains low-boiling substances that act as a local anesthetic on the affected area. It creates an immediate cooling effect at the treated site, alleviates pain for up to 3 minutes, and relaxes the affected area.

Intended Use: Temporary pain relief from muscle bruises, joint sprains, muscle cramps, and other minor sports injuries resulting from impacts, falls, or increased strain, and pain accompanying minor procedures such as venipuncture before inserting a cannula or catheter into a vein.

Indications: Pain associated with muscle bruises, joint sprains, muscle cramps, and other minor sports injuries resulting from impacts, falls, or increased strain, and pain accompanying minor procedures such as venipuncture before inserting a cannula or catheter into a vein.

CONTRAINDICATIONS

Contraindications: Do not spray for an extended period from a short distance to avoid frostbite. Do not use on the face, neck, sensitive body areas, or open wounds. Do not use in case of an allergic reaction to any of the ingredients. Not for use in children under 3 years old. Use during pregnancy and breastfeeding should be consulted with a doctor.

Intended User: Suitable for the general population, children from 3 years of age, and diabetics.

Intended Application Site: Intact skin on affected body areas.

Duration: Temporary effect up to 3 minutes.

Recommended Dose, Instructions for Use: Apply Spophy Coolant Spray by pressing the sprayer onto the skin over the affected area from a distance of 15 cm from the body surface, with 2-3 repeated sprays for a duration of 5 seconds.

Special Safety Requirements for Intended Use: May cause an allergic skin reaction in sensitive individuals. In case of severe redness at the treated site, frostbite, or blistering due to excessive use of the spray, or in case of doubt, always seek medical advice. Protect eyes from spray contact. Do not inhale the spray. Extremely flammable aerosol. The container is under pressure: may burst if heated. Protect from heat, hot surfaces, sparks, open flames, and other sources of ignition. No smoking. Do not puncture or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C. Keep out of reach of children. Any serious incident related to this product should be reported to the manufacturer and the State Institute for Drug Control. Dispose of the container within organized collection systems.

Shelf Life: 30 months. Use no later than the date indicated on the bottom of the container.

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